

Assembly Instructions for the Cannulated PsiFGuard


The Cannulated PsiFGuard® consists of:

- One Electronic Handle Ref. P2HEXXXX (DSG Handle or DSG Connect Handle)
- One Needle XS Ref. P2ND2201 (composed of the cannulated shaft and the sensory stylet)


The Sensory stylet should be screwed completely into the cannulated shaft.

The Needle XS must be used with the electronic handle.

Cannulated PsiFGuard

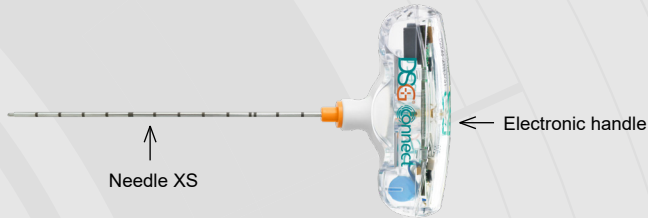

Electronic Handle - P2HEXXXX

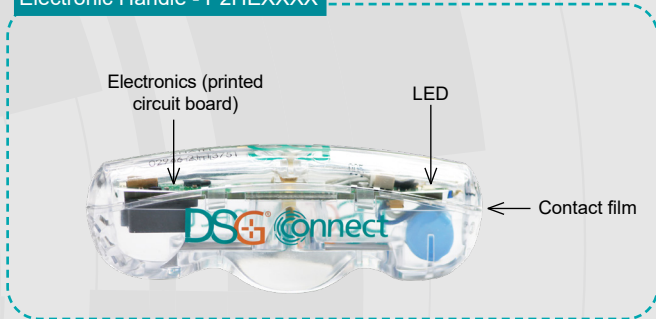


Needle XS - P2ND2201

Cannulated Shaft







Sensory Stylet

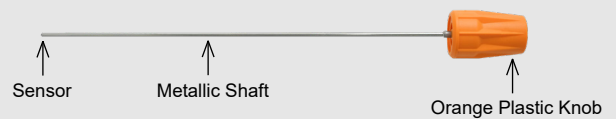


Set-up instructions:

1. Inspect the Needle XS and the Electronic Handle packages for proper integrity.
2. Activate the device by pulling the contact tab out of the Electronic Handle. Activation is confirmed by a beeping sound and green LEDs.
3. Before connecting the Needle XS to the Electronic Handle, check that the Sensory Stylet is fully screwed into the Cannulated Shaft ^[A]. The distal tip of the Sensory Stylet must be visible at the end of the Cannulated shaft.
4. Connect the Needle XS to the Electronic Handle ^[B]. Ensure snap-on connection of the needle and Electronic Handle is secure.
5. The Cannulated PsiFGuard is ready for use ^[C]. Please, refer to the Instruction for Use available on <https://www.spineguard.com/products-and-applications/spine-products/user-manuals/cpg-ifu/>.

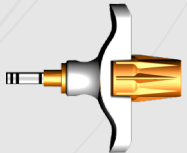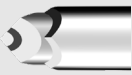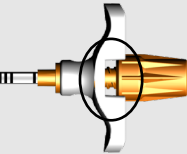[A] Correct screwing of the Sensory Stylet into the Cannulated Shaft	Correct distal tip positioning of the Sensory Stylet into the Cannulated shaft
 <p>Correct</p>	 <p>Correct</p>
 <p>Incorrect</p>	 <p>Incorrect</p>

[B]

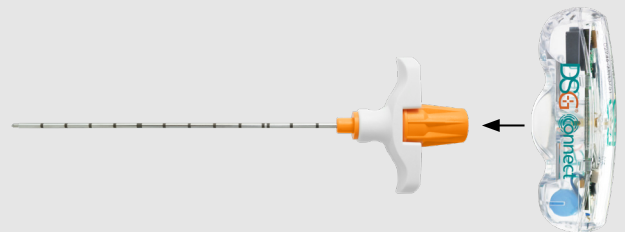

[C]

